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Ropak Kompaktopaker UV

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(EN) Instructions for use

Ropak Kompaktopaker UV

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Ropak Kompaktopaker UV is a light-curing, resin-colored opaque for coating CoCr structures which are veneered with cold-polymerizing pink resin. Thanks to its special color, Ropak Kompaktopaker UV ensures coverage of all metal parts.

2. Indication

Opaque for coating CoCr structures which are veneered with pink resin.

3. Contraindication

The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Safety recommendations and hazard warnings

Signal word:

Danger

Hazard statements:

|      |                                                    |
|------|----------------------------------------------------|
| H225 | Highly flammable liquid and vapour.                |
| H315 | Causes skin irritation.                            |
| H319 | Causes serious eye irritation.                     |
| H317 | May cause an allergic skin reaction.               |
| H335 | May cause respiratory irritation.                  |
| H412 | Harmful to aquatic life with long lasting effects. |

Precautionary statements:

Prevention:

|      |                                                                                                |
|------|------------------------------------------------------------------------------------------------|
| P210 | Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. |
| P261 | Avoid breathing vapours.                                                                       |
| P280 | Wear protective gloves/eye protection.                                                         |

Response:

|                |                                                                                                                                  |
|----------------|----------------------------------------------------------------------------------------------------------------------------------|
| P303+P361+P353 | IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water/shower.                     |
| P305+P351+P338 | IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. |

methyl methacrylate, dipentaerythritol pentaacrylate, 2-(Dimethylamino)ethyl benzoate

Product contains:

5. Storage and durability

Storage:

|             |                                                                                           |
|-------------|-------------------------------------------------------------------------------------------|
| P403+P233   | Store container tightly closed in a well-ventilated place.                                |
| P403+P235   | Store in a well-ventilated place. Keep cool.                                              |
| Durability: | If stored between 4 °C and 25 °C, Ropak Kompaktopaker UV has a shelf life of three years. |

6. Processing

6.1 Important information

To ensure reliable retention of the resin on Ropak Kompaktopaker UV, it is absolutely mandatory to attach retentions to the surface to be veneered.

6.2 Preparation of the structure

After fitting, finishing and polishing the CoCr structure, the area that is to be coated with Ropak Kompaktopaker UV must be sandblasted using 110 µm aluminium oxide (not in the recyclable sandblaster) After sandblasting, clean the CoCr structure with oil-free air or with the steam blaster.

6.3 Processing

**Please note:** Shake Ropak Kompaktopaker UV **well before the use and close the bottle immediately after use!** Apply a thin and uniform layer of Ropak Kompaktopaker UV directly onto the surface using a brush. Small puddles which may result during the application must be removed immediately. Then polymerize as described under item 6.4. When polymerizing, the surface must be aligned with the light source (illuminant) of the polymerization unit. The closer the surface is placed towards the light, the better the quality of polymerization.

6.4 Layer thickness

The layer thickness can be adjusted by applying the material repeatedly and by an intermediate polymerization process of the individual layers. Apply two thin layers of Ropak Kompaktopaker UV to ensure perfect curing and cure each layer separately. Do not touch the varnish between the individual layers.

6.5 Technical data

|                                                |                                                                   |
|------------------------------------------------|-------------------------------------------------------------------|
| Girrbach Labolight LV 1:                       | 5 min. for each layer                                             |
| Ivoclar Spectramat:                            | 5 min. for each layer                                             |
| Dentsply Triad 2000:                           | 2 x 4 min, pay attention to generation of heat                    |
| Kulzer Dentalcolor XS:                         | 90 sec                                                            |
| Polylux-Strahler:                              | 2 x 5 min.                                                        |
| Spectra SL 400:                                | 2 x 4 min                                                         |
| bredent PolyLux:                               | 5 min                                                             |
| bredent bre.Lux PowerUnit/bre.Lux PowerUnit 2: | 180 sec intermediate polymerization, 360 sec final polymerization |

**Please note:** The object should always be placed very close to the light source. The light intensity and hence the effect decreases considerably depending on the distance from the light source and may result in insufficient polymerizing. Dark spots must be avoided.

7. Error sources

7.1 Clouding and discoloration in the resin

**Error:** Ropak Kompaktopaker UV was not cured long enough with the light-curing unit.  
The coat of Ropak Kompaktopaker UV applied was too thick.  
Curing was carried out using an unsuitable light-curing unit.

8. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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