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Kompaktopaker zahnfarben UV

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(EN) Instructions for use

Kompaktopaker UV tooth shades

Before using the product, please read these instruction for careful use.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Kompaktopaker tooth-colored UV is a single-component, light-curing liquid for masking chrome cobalt frameworks prior to adding denture teeth. The neutral color of the Kompaktopaker tooth-colored UV masks the shade of the chrome cobalt alloy and thus simplifies the creation of the desired shade during the fixation of the denture tooth with tooth colored resin.

2. Indication

Opaker from coating CoCr (oder: cast metal) elements which are veneered with tooth-colored resin.

3. Contraindication

Kompaktopaker UV tooth shades **may not** be used for crown and bridge restorations (fixed restorations). This product is not suitable for heat-curing and light-curing veneering resin systems. The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Safety recommendations and hazard warnings

Signal word:

Danger

Hazard statements:

- H225 Highly flammable liquid and vapour.
- H315 Causes skin irritation.
- H319 Causes serious eye irritation.
- H317 May cause an allergic skin reaction.
- H335 May cause respiratory irritation.
- H412 Harmful to aquatic life with long lasting effects.

Precautionary statements:

Prevention:

- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P261 Avoid breathing vapours.
- P280 Wear protective gloves/protective clothing/eye protection/face protection.

Response:

- P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Product contains: methyl methacrylate, dipentaerythritol pentaacrylate, 2-(Dimethylamino)Ethylbenzoat

5. Storage and durability

Storage:

- P403+P233 Store container tightly closed in a well-ventilated place.
- P403+P235 Store in a well-ventilated place. Keep cool.
- Durability: If stored between 4°C and 25°C, Kompaktopaker tooth-colored UV has a shelf life of three years.

6. Processing

Please note: To ensure reliable retention of the resin on Ropak, it is absolutely mandatory to attach retentions to the surface to be veneered.

6.1 Preparation of the object

After placing on, finishing and polishing of the chrome cobalt frame, blast the area that is to be coated with Kompakt-opaker tooth-colored UV with aluminium oxide, grain size 110 µm (units that feature abrasive material circulation are not suitable due to the contamination of the abrasive material since they contaminate the metal surface). After blasting, the metal surface must be cleaned with oil-free compressed air or a steam-jet unit.

6.2 Processing

Please note: Shake bottle well prior to the use and close it immediately after the use.

Apply Kompaktopaker tooth-colored UV thinly and evenly onto the dust- and grease-free metal surface using a brush. Small puddles that result during the application must be removed immediately. Then cure according to the polymerization times (see item 6.4).

6.3 Layer thickness

The layer thickness can be adjusted by applying the material several times and by carrying out intermediate polymerization of the individual layers. Apply **two thin layers** of Kompaktopaker tooth-colored UV **to ensure perfect curing** and cure each layer separately. Do not touch the varnish surface between the individual layers.

6.4 Polymerization

Please note: During light-curing ensure that the varnish-coated surface is as close as possible to the light outlet of the light-curing unit. Dark spots must be avoided. Depending on the respective polymerization unit, cure the wetted surface as follows.

Technical data:

Girrbach Labolight LV 1:	5 min. for each layer
Ivoclar Spectramat:	5 min. for each layer
Dentsply Triad 2000:	2 x 4 min., pay attention to generation of heat
Kulzer Dentalcolor XS:	90 sec
Polylux-Strahler:	2 x 5 min.
Spectra SL 400:	2 x 4 min.
bredent Polylux:	5 min.
bredent bre.Lux PowerUnit/ bre.Lux PowerUnit 2:	180 sec intermediate polymerization, 360 sec final polymerization

Please note: The object should always be placed very close to the light source. The light intensity and hence the effect decreases depending on the distance from the light source and may result in insufficient polymerizing. Dark spots must be avoided.

7. Error sources

7.1 Clouding and discoloration in the resin

Error: Kompaktopaker tooth-colored UV was not cured long enough with the light-curing unit. The coat of Kompaktopaker tooth-colored UV applied was too thick. Curing was carried out using an unsuitable light-curing unit.

8. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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