Abraso- Schwabbel Acryl

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Abraso- Buff Acryl

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Abraso-Buff Acryl is a polishing brush consisting of three rows of bleached Chungking bristles and two textile layers. The textile layers retain the pumice polishing paste for a longer time so that less polishing paste needs to be applied.

2. Indication

The Abraso-Buff Acryl is used for prepolishing all dental acrylics.

3. Contraindication

Abraso-Buff Acryl is not suitable for polishing metals.

4. Processing

The acrylic surface to be polished is prepared in the usual way. The Abraso-Buffs Acryl are turned on the spindle of the polishing motor. The spindle retainer of Abraso-Buff Acryl is conical. It must be ensured that the conical spindle retainer of Abraso-Buff Acryl is turned on the spindle with the large opening first. The polishing brush should only be fastened handtight on the spindle – risk of fracture! If adequate hold of the polishing brush is not ensured, the polishing brush must be fastened on the spindle again or – in case of worn-out thread – a new spindle must be mounted. Then polish at a speed of 3000 rpm. Clean the acrylic surface in rotating movements using pumice powder and DentaClean pumice disinfectant, (REF 520 0099 8), or pumice polishing paste, (REF 520 0016 0) until the surface is free from grinding scratches. To obtain perfect high lustre, clean the denture and polish with the high lustre buff Acryl, (REF 350 0082 0) and Abraso-Star K50, (REF 520 0016 1). When working with the polishing motor, do not polish the same area for a longer period; heat is generated. This may result in damage to the acrylic.

5. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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