Gebruiksaanweisung

Abraso-Soft Acryl

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Instructions for use

Abraso-Soft Acryl

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

The center of Abraso-Soft Acryl is made of fibre fleece and the outside is made of bleached Chungking bristles. These materials take up more pumice powder and hold it over an extended period. The fleece reduces the friction heat.

bredent

Abraso-Soft Acryl is available in diameters of 50 and 80 mm and enables easy access to all areas.

2. Indication

Abraso-Soft Acryl is suitable for prepolishing all dental acrylics.

3. Processing

The acrylic surface to be polished is prepared in the usual way. Turn the Abraso-Soft Acryl onto the spindle of the polishing motor. The spindle clamping device of Abraso-Soft is conical. It must be ensured that the conical spindle clamping device of Abraso-Soft Acryl is turned onto the spindle with the large opening first. The polishing brush is only hand-screwed onto the spindle – risk of breakage! If the polishing brush is not firmly held on the spindle, attach the brush to the spindle again or mount a new spindle if the spindle thread is damaged or worn out. Then polish at a speed of 3000 rpm. Apply pumice powder and Dentaclean pumice disinfectant (REF 520 0099 8) or pumice polishing paste (REF 520 0016 0) to the acrylic surface in rotating movements until all grinding marks are removed. To achieve perfect high lustre, clean the denture and polish with the high lustre buff Acryl (REF 350 0082 0) and Abraso-Star K50 (REF 520 0016 1). When working with the polishing motor, do not polish the same area for a longer period; heat is generated which may result in damage to the acrylic.

4. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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