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000816EX- 20090902

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# High luster buff Acryl

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

## 1. Indication range

The high luster buff Acryl is suitable for high luster polishing all dental acrylics.

## 2. Indication

High luster polishing of dental acrylics mounted to the polishing unit.

## 3. Processing

Thanks to special pretreatment of the linen layers truing with a sharp object is omitted and the associated formation of fuzz is avoided.

The acrylic surface to be polished is prepared in the usual way. Turn the high luster buff Acryl onto the spindle of the polishing motor. The spindle intake of the high luster buff Acryl is conical. It must be ensured that the conical spindle intake of the high luster buff Acryl is turned onto the spindle with the label resp. large opening first. The polishing brush must be carefully screwed hand tight onto the spindle – risk of breakage! If safe hold of the polishing brush is not ensured, put the polishing brush on the spindle again or mount a new spindle in case of wear of the spindle thread.

This buff made of a special fabric allows to achieve perfect high luster in conjunction with Abraso-Star K50 polishing paste (REF 520 0016 1).

The special layers of linen (fabric) of the buff produce a silky soft surface. Even at a speed of 3000 rpm excessive generation of heat is avoided.

## 4. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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GmbH & Co. KG · Weissenhorner Str. 2 · 89250 Senden · Germany