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Optiguss

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(EN) Instructions for use

Optiguss

Before using the product, please read these instructions for use carefully!

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Optiguss are two special varnishes for different application of layers (5 µm und 10 µm) to smoothen sur-faces of wax models prior to investing.

2. Indication

Sealing and smoothing of surfaces of wax models for of crown and bridge restorations as well as for cast partial dentures.

3. Contraindication

Based on the current state of knowledge there are no known contraindications.

4. Safety recommendations and hazard warnings

Optiguss micro

Signal word:

Danger

Hazard statements:

H225 Highly flammable liquid and vapour.  
H361d Suspected of damaging the unborn child.  
H335-H336 May cause respiratory irritation. May cause drowsiness or dizziness.

Precautionary statements:

Prevention:

P201 Obtain special instructions before use.  
P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
P233 Keep container tightly closed.  
P261 Avoid breathing vapours.  
P280 Wear protective gloves eye protection.  
P281 Use personal protective equipment as required.

Response:

P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.  
P308+P313 IF EXPOSED OR CONCERNED: Get medical edvice/attention.

Disposal:

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.  
pentan-3-one, 2-(2-methoxyethoxy)ethanol

Product contains:

Optiguss macro

Signal word:

Danger

Hazard statements:

H225 Highly flammable liquid and vapour.

Precautionary statements:

Prevention:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
P233 Keep container tightly closed.  
P280 Wear protective gloves/eye protection.  
P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

Disposal:

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

Brush cleaner

Signal word:

Danger

Hazard statements:

H225 Highly flammable liquid and vapour.  
H319 Causes serious eyey irritation.  
H336 May cause drowsiness or dizziness.  
EUH066 Repeated exposure may cause skin dryness or cracking.

Precautionary statements:

Prevention:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
P233 Keep container tightly closed.  
P261 Avoid breathing vapours.  
P280 Wear protective gloves/protective clothing/eye protection/face protection.  
P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Product contains:

5. Storage and durability

Storage:

P403+P235 Store in a well-ventilated place. Keep cool. Optiguss must be stored at room temperature under dry conditions and protected against light.

6. Processing

6.1 Preparation of the object

The wax model is produced in the usual way.

6.2 Processing

Optiguss varnish is available in two thicknesses of micro 5 µm and macro 10 µm. Add Optiguss into the Optiguss dishes. Then use the special brush, which is contained (three different sizes) in the set, to the wax model. By applying several layers (coats) of Optiguss, wax models can be reinforced in a controlled manner (e.g. at the contact points). The processing time for Optiguss macro is approx. 45 seconds and the processing time for micro is approx. 90 seconds. Then do not move the brush across the Optiguss coat to prevent the varnish from forming strings. Optiguss needs to dry at least 10 minutes before additional layers are applied.

6.3 Investing

After the application, Optiguss needs to dry at least 10 minutes. Do not touch the surface with your fingers. Do not use wax surface tension reducing agent since Optiguss acts similar to such materials. If wax surface tension reducing agent is to be applied to the interior of crowns, this should be done prior to the use of Optiguss. Then apply Optiguss to the completely dry surface as described under item 6.2. Afterwards invest the object in the usual way.

6.4 Cleaning brushes

The brushes can be quickly and easily cleaning in the brush cleaner jar using the brush cleaner. The liquid can be used several times.

6.5 We recommend processing at a well-ventilated workplace.

7. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff. These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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bredent GmbH & Co. KG