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Instructions for use  
SERACOLL UV

Before using the product, please read these instructions for use carefully!

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

SERACOLL UV is a light-curing adhesive.

2. Indication

SERACOLL UV is used for connecting models made of modelling wax, light-curing waxes and brush resins as well as for smoothing and fixing models in the copy milling technique.

3. Contraindication

SERACOLL UV is not suitable for oral use.

4. Hazard and safety information

Signal word:

Danger

Hazard statements:

- H225 Highly flammable liquid and vapour.
- H315 Causes skin irritation.
- H317 May cause respiratory irritation.
- H319 Causes serious eye irritation.
- H335 May cause respiratory irritation.
- H412 Harmful to aquatic life with long lasting effects.

Precautionary statements:

Prevention:

- P210 Keep away heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P233 Keep container tightly closed.
- P261 Avoid breathing vapours.
- P280 Wear protective gloves/eye protection.

Response:

- P303+P361+P533 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Product contains: methyl methacrylate

5. Storage and durability

Storage:

- P403+P235 Store in a well-ventilated place. Keep cool.

6. Processing

Prepare the model in the usual way. Separate the model and ensure that the separating gap is as thin as possible; if required reduce the gap by applying additional material. It is also possible to glue (bond) several separating gaps at one time; in such cases the optimal width of the separating gap must be ensured. After checking and contouring the crown margins, place the wax-up back onto the model. Use a probe to take up one drop of SERACOLL UV and let it flow into the gap (optimal size >0.3 mm). Then cure from each side for 10 seconds in a standard UV light-curing unit or using an LED lamp. After the application of SERACOLL UV, the model should never be placed in a UV light-curing unit which has heated up owing to frequent usage since the heat inside the system may affect the stress properties (freedom/absence of stress); let the unit cool down first. Attach the sprues to the object, remove the wax-up from the model only after the wax has hardened (cooled down). SERACOLL UV is also suitable for stress-free fixation of sprues for light-curing to the object.

7. Symbols

7.1 Symbols according to DIN EN ISO 15223-1

- REF Reference number
- LOT Lot number
- Can be used until YYYY-MM
- Keep away from sunlight
- Manufacturer
- Attention!

7.2 Symbols according to GHS regulation

- GHS02, flame
- GHS07, Exclamation mark

7.3 Symbols according to NFPA (USA)

- Hazard diamond

8. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff.

Only original tools and components may be used for processing. These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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