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Qu-resin

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(EN) Instructions for use

Qu-resin

Before using this product, please read the instructions for use carefully!

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Qu-resin is a quick-setting, self-curing repair resin based on diacrylate. Qu-resin is offered as a paste-like, two-component material in a double cartridge (mixing ratio: 1:1) and is available in a tooth-colored (Qu-resin dentin) and a pink version (Qu-resin pink).

2. Indications

- Repair and extension of resin dentures (PMMA)
- Intraoral connection of implant impression abutments

Qu-resin is suitable for intra- and extraoral use.

2.1 Patient target group

Adults who require dental restoration

2.2 Intended users

The product may only be used by dentists, dental technicians and adequately trained dental staff.

3. Contraindication

The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Hazard and safety information

Hazard information:

H412	Harmful to aquatic organisms, with long lasting effects.
EUH204	Contains isocyanates. May cause allergic reactions.
EUH208	Contains dibenzoyl peroxide. May cause allergic reactions.

Safety information:

Prevention:

P273	Avoid release into the environment.
------	-------------------------------------

5. Storage and durability information

Qu-resin has a durability of 18 months if storage between 15°C to 23°C is ensured.

Disposal:

P501	Disposal of contents / of the container in accordance with / regional / national / international regulations.
------	---

6. Processing / Use

6.1 Preparing

The components are applied from the double cartridge using a suitable dosing device (e.g. REF 320000440). Remove the cartridge cap (by turning it) and discard material until both pastes are dispensed evenly. To ensure perfect mixing of the material, use only the enclosed mixing cannulas. The mixing cannula in use is used to seal the cartridge until it is used again.

6.2 Repair or extensions

Use 110 µm aluminium oxide to sandblast the area to be repaired or extended, clean with oil-free compressed air and wet with Qu-connector. Harden Qu-connector in a suitable light-curing device (e.g. wavelength range: 370 - 500 nm, bre.Lux Power Unit 2) for 90 seconds (observe the instructions for use of Qu-connector). The curing time of Qu-resin is 3 minutes at room temperature. Ethyl alcohol can be used to clean the resin surface.

6.3 Connecting impression abutments

Intraoral connecting of implant impression abutments with Qu-resin allows achieving more accurate impressions. Once the impression abutments have been screwed on, dental floss is tension crosswise between the abutments. Then Qu-resin (pink or dentin) is applied to the dental floss and the impression abutments to connect them. The impression is taken in the usual way after approx. 3 minutes.

7. Technical data

Composition	
Acrylates	~ 61 %
Fillers	~ 37 %
Initiators, stabilizers, pigments	~ 2 %

Properties according to DIN EN ISO 10477 (typical values)	
Flexural strength	100 MPa
Modulus of elasticity	3.600 MPa
Water absorption	22 µg/mm3
Solubility	0,5 µg/mm3

9. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff. These instructions for use need to be handed over to the user as an accompanying document for insertion or further processing of the product. Only original tools and components may be used for processing.

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

The user and / or patient are obliged to report any serious incidents that have occurred in relation to the device to the manufacturer and / or the competent authority of the Member State in which the user and / or patient is registered.

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