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009306EX-20190326

Qu-connector

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(EN) Instructions for use

Qu-connector

Please read these instructions for use carefully before using the product.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Product description

Qu-connector is an acrylate-based light-curing adhesive. Qu-connector is used to create an adhesive bond between denture base resins (PMMA) and Qu-resin.

2. Indication

Conditioning of acrylic dentures (PMMA) for repairs and extensions with Qu-resin.

2.1 Target patient groups

Adults requiring dentures

2.2 Intended user

The product must only be used by dentists, dental technicians and adequately trained dental staff.

3. Contraindication

Qu-connector is not suitable for intraoral use.

If intolerance is suspected, the product must not be used until allergy tests that confirm the absence of an allergy have been completed.

4. Hazard and safety information

Signal word:

Danger

Hazard statements:

H225	Highly flammable liquid and vapour.
H315	Causes skin irritation.
H317	May cause an allergic skin reaction.
H319	Causes serious eye irritation.
H335	May cause respiratory irritation.
H361f	Suspected of damaging fertility.

Precautionary statements:

Prevention:

P201	Obtain special instructions before use.
P210	Keep away from heat / sparks / open flames / hot surfaces. No smoking.
P261	Avoid breathing vapours.
P280	Wear protective gloves / protective clothing / eye protection / face protection.
P281	Use personal protective equipment as required.

Reaction:

P303+P361+P353	If on skin (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.
P305+P351+P338	If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P308+P313	If exposed or concerned: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash before reuse.

Product contains:

Methyl methacrylate, diphenyl (2,4,6-trimethylbenzoyl) phosphine oxide, pentaerythritol triacrylate, acrylate polymers

5. Storage and shelf life information

Storage:

P403+P233	Store in a well ventilated place. Keep container tightly closed.
P403+P235	Store in a well ventilated place. Keep cool.

Shelf life:

Qu-connector can be stored for 2 years at a temperature between 15 °C and 23 °C.

6. Processing / handling

6.1 Preparation

Before creating the adhesive bond, sandblast the affected areas with 110 µm aluminium oxide and clean with oil-free compressed air. The surfaces must be free of dust and grease.

6.2 Application

Apply Qu-connector and cure for 90 seconds in a suitable light-curing unit (wavelength range 370 - 500 nm, e.g. in a bre.Lux Power Unit 2). Then process with Qu-resin following the instructions for use.

7. Technical data

Composition	
Acrylate	~ 85%
Filler, initiators	~ 25%

8. Other information

The product must only be used by dentists, dental technicians and adequately trained dental staff. For the product processing, the instructions for use must be handed to the practitioner as an accompanying document for the insertion or further processing. Original tools and components must be used for processing.

These instructions for use correspond to the current state of the art and our own experience. The product must only be used in the indication described in section 2. The user is responsible for the use of the product. No liability is accepted for incorrect results, as the manufacturer has no influence on the processing of the product. Any claims for damages that are made nonetheless may only be made up to the value of our products.

The user and / or the patient is / are obliged to report any serious incidents occurring in connection with the product to the manufacturer and / or the responsible authority of the Member State in which the user and / or the patient resides.

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bredent GmbH & Co. KG