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Instructions for use

beautyssetup Aesthetic add-on wax

Before using the product, please read these instructions for use carefully!

1. Product description

beautyssetup aesthetic add-on wax is a wax for set-ups and the fabrication of models and burns out without leaving any residue.

2. Indication range

beautyssetup aesthetic add-on wax is used to prepare aesthetic set-ups. It is particularly suitable for the casting technique (CoCr work) since it burns out without any residue. beautyssetup aesthetic add-on wax is especially intended to be used for the visio.lign veneer system. Missing anatomical characteristics can be added using beautyssetup aesthetic add-on wax. Individual coloring of areas to be completed is achieved by mixing beautyssetup aesthetic add-on wax light (REF 430 0030 0) and dark (REF 430 0031 0).

3. Indication

Completing, modeling and setting up aesthetic set-ups of veneers and artificial teeth for oral try-in.

4. Contraindication

Not suitable for permanent use in the mouth.

5. Application

Based on the standard procedure, beautyssetup aesthetic add-on wax is melted with wax modelling instruments, applied and processed just like other classic modelling waxes.

6. Storage information

Protect beautyssetup aesthetic add-on wax against sunlight and store at 0° C – 25° C.

7. Symbols

	Observe instructions for use
	Lot number
	Temperature limit
	protect against sunlight

8. Additional information

The information contained in these instructions for use is always updated according to the latest knowledge and experience. Therefore we recommend reading the instructions for use again before using a new package. These non-binding user recommendations are based on our own experience. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since bredent GmbH & Co.KG does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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GmbH & Co.KG · Weissenhorner Str. 2 · 89250 Senden · Germany