visio.sil

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(EN) Instructions for use

visio.sil

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

visio.sil is a transparent, addition-curing silicone for keys supplied in the 50 ml automix cartridge.

2. Indication / application range

visio.sil was especially developed for the use in the pre-impression technique with UV-curing materials. When used with the visio.lign veneering system, it allows swift and economic processing based on light-curing through the key material. It is particularly suitable to „lengthen“ tooth necks or for direct preparation of interdental spaces.

3. Contraindication

visio.sil is not suitable for oral use.

4. Safety recommendations and hazard warning

visio.sil may only be used by dental technicians and qualified dental staff experienced in processing silicones. visio.sil is exclusively intended for dental-technical applications (for finishing).

Avoid contact with skin and eyes and do not swallow the material.

Please refer to the safety data sheet for additional safety recommendations concerning the product.

The lot numbers of all components in use must be documented to ensure traceability and to handle complaints.

5. Storage and durability

visio.sil should be stored between 4 °C and 25 °C. Storage of the material below a room temperature of 21 °C results in a delay of the setting reaction if an extended processing time span is required.

6. Processing

Application

visio.sil is directly applied from the automix cartridge system. The green mixing cannula (REF 320 00 45 7) is used for applying the material. The discharge opening of the mixing cannula should be in the material during the application to avoid entrapment of air. Curing is carried out within 4 minutes.

To reinforce the key, in particular for large-sized reconstructions, a kneading silicone, for example haptosil D (REF 540 0118 0), should be added into the vestibular fold of the model and visio.sil should be applied to the occlusal area without any delay.

visio.sil fix, a very precise, addition-cured silicone (REF 540 0130 0) is recommended for sealing interdental spaces.

7. Technical data

Material: addition-curing silicone elastomer

Shore A hardness: 60

8. Symbols

European conformity marking

Reference number

Can be used until YYYY-MM

Lot number

Attention!

Temperature limits
Lower limit: 4 °C
Upper limit: 25 °C

Protect against sunlight

Manufacturer

9. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff. Only original tools and components may be used for processing.

All parts must be disinfected prior to the use in the patient's mouth (in situ). Only disinfectants for dental instruments may be used, e.g. Dürr ID 212 forte. The manufacturer's instructions on the use of the disinfectant must be observed. These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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bredent GmbH & Co. KG