Gebruuchsanweisung

K-Primer

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Instructions for use

K-Primer

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product
K-Primer is a bonding agent for bonding ceramic and resin respectively metal and resins if silicization has been previously carried out.
2. Indication
a) Conditioning of ceramic veneers (silicate, glass or feldspar ceramics and lithium disilicate ceramic) frameworks (precious metal-free) to be bonded, veneered or repaired with composite or resin (methacrylate-based).
b) If silicization has been previously carried out, the product is used for conditioning of metal frameworks for crowns, bridges, secondary structures, cast metal and abutments to be veneered with or bonded to composite or resin (methacrylate-based) later on.
c) If silicization has been previously carried out, the product is used for conditioning of oxide ceramic veneers (e.g. made of zirconium dioxide) for crowns, secondary structures and abutments to be bonded or veneered with composite or resin (methacrylate-based) later on.
The product is suitable for intraoral and extraoral use.

3. Contraindication
Conditioning of resin dentures, acrylic teeth or natural teeth (enamel or dentin) resp. substructure materials for ceramic veneering.

4. Hazard and safety information	
Signal word: Danger	
Hazard statements: H225 Highly flammable liquid and vapour. H319 Causes serious eye irritation. H336 May cause drowsiness or dizziness. EUH066 Repeated exposure may cause skin dryness or cracking.	
Precautionary statements: Prevention: P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Avoid breathing vapours. Wear protective gloves / eye protection.	
Response: P303+P361+P353 IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.	
Storage: P403+P235 Store in a well-ventilated place. Keep cool. Product contains: butanone	

5. Storage and durability
The product must be protected against heat and stored at 4 - 25° C. K-Primer must not be used after the expiration date any longer.

6. Processing / Use
Attention: Do not add K-Primer onto metal or ceramic mixing plates or dishes since K-Primer will react with them and lose its effect!
6.1 Conditioning of ceramic frameworks
Sandblast the veneer, crown or bridge at a pressure of 3 to 4 bars using 50 - 110 µm aluminium oxide roughen it using a diamond-coated grinder. **Note:** Contamination with oil, separating agent or moisture (e.g. by touching the object with the fingers) must be avoided. Do not clean with a steam jet.
A some drops of K-Primer onto a mixing plate or in a mixing dish, apply to the framework using a disposable brush and let it dry for approx. 30 sec.

6.2 Conditioning of metal and zirconium oxide frameworks
Sandblast the framework at max. 2 bars (activation) using 110 µm aluminium oxide and clean with air or a brush. **Note:** Contamination with oil, separating agent or moisture (e.g. by touching the object with the fingers) must be avoided. Then silicize the surface, e.g. by using Silano Pen (bredent) or Rocatec (3M ESPE). Add drops of K-Primer onto a mixing plate or in a mixing dish, coat the framework completely using a disposable brush and let it dry for approx. 30 sec.

6.3 Applying veneering material
First apply a thin layer of crea.lign of flowable composite or opaquer immediately after conditioning and polymerize in accordance with the respective instructions for use. Adequate wetting of the surface must be ensured.

6.4 Intraoral repair of damaged crowns or bridge veneers made of ceramic
Place rubber dam. Sandblast framework and veneer surface carefully or roughen with diamond-coated grinder (ensure dry condition). Use a brush to apply a rich quantity and a thick coat of K-Primer to the framework and adjacent veneer (transition area), wait 15 sec. and coat the framework with opaquer / adhesive liner and polymerize in accordance with the manufacturer's instructions. Use veneering material to complete the veneer and polymerize.

Note: it is mandatory to use rubber dam and avoid direct contact with the mucous membrane. In case of contact, rinse immediately with water.

7. Symbols	
7.1 Symbols according to DIN EN ISO 15233-1	
	Attention!
	Can be used until YYYY-MM
	Lot number
	Temperature limits (upper and lower limit, see durability and storage)
	Manufacturer

7.2 Symbols according to GHS regulation	
	GHS02, Flamme
	GHS07, Ausrufezeichen

7.3 Symbols according to NFPA (USA)	
	Hazard diamond

8. Additional information
The product may only be used by dentists, dental technicians and adequately trained dental staff. Only original tools and components may be used for processing. The lot numbers of all components should be recorded to ensure traceability and to handle complaints. These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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bredent GmbH & Co. KG · Weissenhorner Str. 2 · 89250 Senden · Germany